



SAVE CHAIR TIME

Quickcheck offers the ability to make precise, accurate and fast chair side adjustments when seating any type of restoration. Quickcheck saves time for you, your staff and your patients, helping your office maintain its busy schedule, while insuring the highest levels of accuracy and fit. Don't let minor adjustments take major time, use Quickcheck.

VACALON

HOW IT WORKS. Locating contact points, unwanted or wanted, while seating restorations or checking fits is crucial for any dental office. As Quickcheck is sprayed onto any stone model, tooth or a restorative surface, wet or dry, it leaves an ultra smooth colored powder layer. Then when you seat a restoration, implant or close a bite the powder transfers to the unsprayed surface and marks the sprayed surface. This clearly reveals the contact points allowing you to make precise adjustments with the proper guidance and location.

THE PERFECT FIT, EVERY TIME. Quickcheck is best known for assuring a proper fit on crowns, bridges and partials by locating high spots and binding areas. However, Quickcheck is a precision instrument that gives doctors the ability to see beyond what is ordinarily visible. Quickcheck can leave a powder layer as thin as 5 microns, allowing doctors to achieve the highest levels of accuracy in a minimal amount of time. Whether fixed or removable, Quickcheck will help complete tasks faster and more accurately.

SAVING TIME. Whether it is 45 minutes or 45 seconds your time is valuable, and time is something every dental office struggles to save. The minutes add up and at the end of the day wasted time can stop you from reaching your full potential. By implementing the use of Quickcheck in your office you will be able to quickly achieve the best in fit and accuracy when performing many tasks, especially seating all types of restorations. Quickcheck gives you the ability to reduce chair side adjustments saving time for you, your staff and your patients.

NEW PATENTED SPRAY BUTTON. A new patented delivery system insures a smoother, more controlled spray with faster drying and more even coatings. It offers true instantaneous drying and prevents any dripping, running or clogging. This system can be used with or without the use of an extension tube.



SAFE FOR USE IN DENTAL OFFICES. Quickcheck is made using our proprietary formulation, which can only be found in our product. Manufactured in the USA, under stringent guidelines, Quickcheck is listed with the FDA and safe for intra-oral use.

MORE PRODUCT FOR LESS. Quickcheck contains over three times more product in each can than most competitive products at about the same price, making it a very economical purchase for any office. At 75 grams by weight and 225 ml by volume, Quickcheck offers up to 500 applications per can. This is just one reason why Quickcheck can be one of the biggest money savers for your office.

APPLICATIONS

- Seating Crowns
- Seating Bridges
- Locating Undercuts
- Adjusting Wax-Ups
- Seating Partials
- Fine Tuning Implants
- Equilibrating Models
- Quality Control
- Adjusting Precision Attachments
- Scan Spray with Cad/Cam Systems
- Seating Inlays
- Locating Sore Spots on Dentures
- Temporary Splints
- Highlighting Contour Lines on Stone Models
- Seating Overdentures
- Working with Custom Attachments
- Seating Maryland Bridges
- Finishing Custom Bars

MATERIALS

- Any type of alloy, including gold
- Acrylics
- Composites
- Zirconium and other Cad/Cam materials
- Pressable Ceramics
- All Porcelains
- Natural Tooth Surface
- Gypsum
- Plastics



CROWNS, BRIDGES, INLAYS AND ONLAYS

1. Shake can well before use 2. a) Lightly spray enough Quickcheck to cover the entire inside of the restoration or for checking occlusal contacts spray the outside of the restoration b) Or lightly spray enough Quickcheck to cover the prepped tooth or teeth 3. Seat the restoration. If checking occlusal surfaces have patient bite down 4. Remove the restoration 5. Check inside of the restoration for high spot markings, check the preparation for markings or where Quickcheck transferred from the prep, indicating premature contacts 6. Using the information obtained from checking the restoration with Quickcheck, adjust proper areas with desired rotary instrument 7. Repeat as needed, clean Quickcheck from the restoration and polish before final seating.



PARTIALS

1. Shake can well before use 2. Lightly spray enough Quickcheck to cover the palatal area and the clasps of the framework 3. Seat framework 4. Remove framework, check for marked areas on the framework and clasps that indicate binding areas or tight spots 5. Adjust high spot or binding areas with desired instrument 6. Repeat as needed 7. Clean Quickcheck from partial and remove any scratches made by adjustments before final seating



INTERPROXIMAL AREAS

1. Shake can well before use 2. Lightly spray enough Quickcheck to cover the mesial and distal areas of the restoration 3. Seat restoration, checking for proper contact points 4. If contact points are too tight, remove crown and adjust as needed with desired rotary instrument 5. Repeat as needed until desirable contacts are obtained 6. Clean Quickcheck from restoration and polish area to remove any scratches made by adjustments before final seating



ADJUSTING DENTURES (WHITE QUICKCHECK)

1. Shake can well before use 2. Lightly spray White Quickcheck on the entire palatal area or suspected sore spot area 3. Seat denture, have patient bite into centric position, remove denture and check for high spot areas 4. Adjust any high spot area with proper bur 5. Repeat procedure until high spot areas are removed 6. Clean Quickcheck from denture, polish and pumice adjusted area before final seating



CAD/CAM SYSTEMS (WHITE QUICKCHECK)

1. Shake can well before use 2. Always dry prepped tooth before spraying Quickcheck 3. Lightly coat the prepped tooth or die that is to be scanned with White Quickcheck 4. Scan prepped tooth 5. Clean prepped tooth area



SPECIAL OFFER

For a limited time receive **FREE SHIPPING** when you order our introductory kit, which includes 1 can of each color, for only **\$49.99**. Order online at **www.quickcheckdirect.com** or call **888.577.9340** today and start saving valuable chair time.

888.577.9340

VACALON

QUICKCHECKDIRECT.COM



QUICKCHECK INDICATING SPRAY is an indispensable work aid that is a must have for every dental office and operatory. One of its many uses is giving doctors the ability to pinpoint, and then relieve, the exact areas on the inside of a restoration that cause misfit crowns or rocking bridges. These areas are commonly caused by undercuts, micro-bubbles or inadequate fits. Quickcheck is compatible with any type of dental restorative material and can be used on any type of restoration. Quickcheck will help you reduce the chance of remakes and achieve the perfect fit every time.

- Saves time, labor and money by quickly locating trouble areas and helping to achieve a precise fit
- Useful with crowns, bridges, partials, dentures or to locate occlusal contacts
- Quick instantaneous application dispenses a micro-fine layer of colored powder
- Economically sized 75 gram can contains over 3 times more material than most other brands
- New patented spray button offers a smoother more controlled spray, with faster drying and even coatings without the use of an extension tube.
- Contains no CFC's
- Easily removed with a brush and water, steamer, or moistened gauze
- White works great with all-ceramic restorations and as a scan spray with most Cad/Cam systems



QUICKCHECK GREEN
75 grams • Item 10050
\$18.99



QUICKCHECK RED
75 grams • Item 10060
\$18.99



QUICKCHECK WHITE
75 grams • Item 10070
\$18.99

888.577.9340

VACALON

QUICKCHECKDIRECT.COM