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Operating Instructions
Pin drilling unit G1 Concept

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We are pleased that you decided to buy a highly developed piece of equipment from SCHICK and would like to wish you every success when working with your new G1 Concept pin drilling unit.

We wrote these operating instructions to enable you to get accustomed to your new piece of equipment and to provide you with the correct operating and maintenance instructions.

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Index

1. List of contents - accessories
2. Range of applications
3. General information
Caution !
4. Picture G1 Concept -
operating elements
5. Setting up
6. Operating the unit
- 6.1 Safety Information
7. Exchanging the tool
8. Maintenance and care
9. Possible faults
10. Technical data
11. Declaration of conformity

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

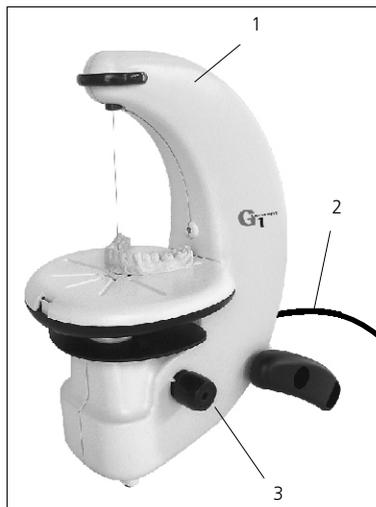
1. List of contents - accessories/spare parts

G1 Concept Art.No.: 1340

Pin drilling unit complete with point marking laser (1)
 mains cable (2)
 drill depth adjustment (3)
 hexagon key Sw06 (4) W602 000 500
 hexagon key Sw03 (5) W602 000 300
 finger protection

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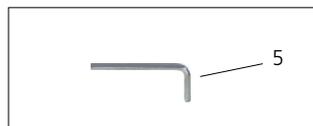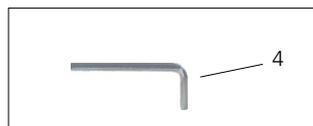Laserklasse: 2

2. Range of Applications

The recently developed pinhole drilling unit from Schick Dental's Concept range with its well-known futuristic design and ergonomic handling is a must for every modern dental laboratory.

Select the ideal working position by simply adjusting the stand, and you can choose whether to work in the standing or sitting position. A laser light source guarantees exact positioning

and drilling under all lighting conditions and with all model materials. With the novel Auto-Lock-System -ALS-, changing drills couldn't be easier - and no tools are needed. The drill depth gauge can be comfortably and steplessly adjusted via an external adjusting screw.

Connection to an external suction device ensures optimal cleanliness during and after drilling.

Removal of the drilling dust leaves clean drill holes on the model and device. No subsequent preparation is needed. The pins can be inserted immediately.

Conditions of environment: - interior 10° - 40°C

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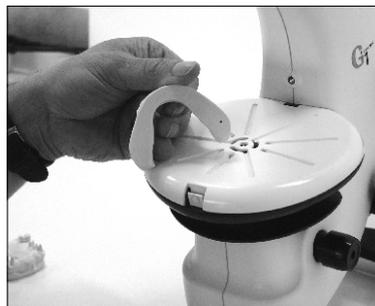
3. General Information

- 3.1 Ascertain that your mains supply coincides with the data in the rating plate.
- 3.2 The pin drilling unit G1 Concept is not suitable for the following applications:
 - in areas where there is a risk of explosion
 - on patients
 - for working on moist materials
- 3.3 Insure that all regulatory requirements are observed during use
 - always wear protective glasses
 - take care for good illumination
 - use dust suction

Caution

- Repairs and other technical procedures must only be carried out by suitable qualified personnel, authorized by SCHICK.
- SCHICK do not guarantee the pin drilling unit G1 Concept should it not have been used in accordance with the operation instructions.

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

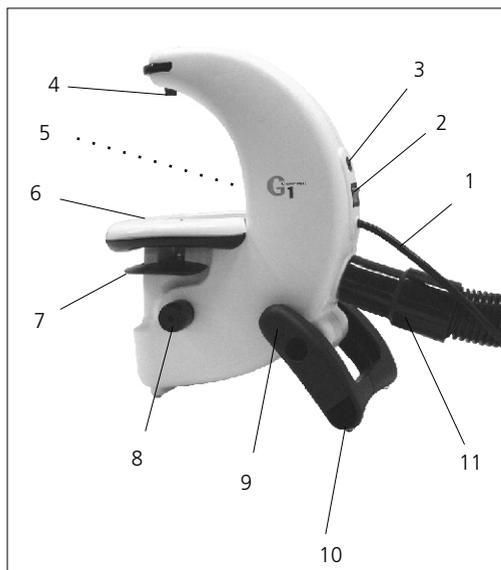
4. Picture G1 Concept - operating elements

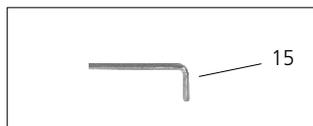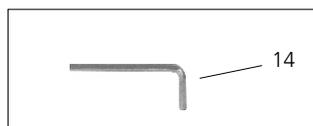- 1 - mains cable
- 2 - mains switch
- 3 - fuse 3,15A/T 220V (6,3A/T 100V-115V)
- 4 - point laser
- 5 - power control lamp
- 6 - drilling table
- 7 - operation plate
- 8 - adjusting screw for drilling depth gauge
- 9 - adjusting hexagon screw for table angle
- 10 - adjustable stand
- 11 - extractor socket for external dust suction
- 14 - hexagon key SW06
- 15 - hexagon key SW03

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


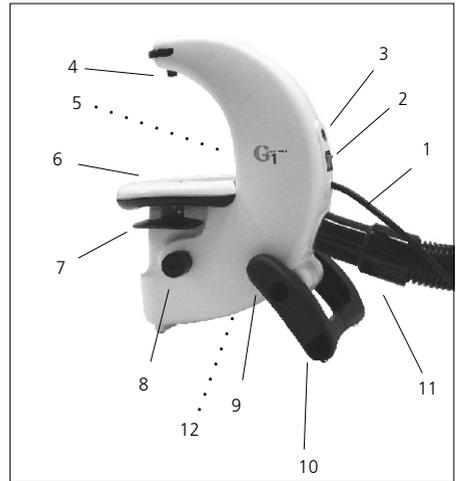
5. Setting up

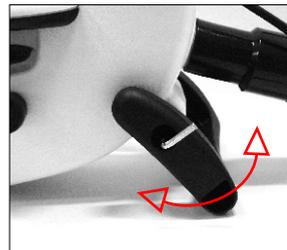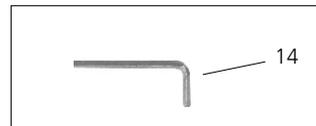- Unpack the G1-Concept and check if complete
- Check whether power data with the information on the identification plate (12) corresponds
- Power plug (14) in mains socket or in the external suction socket.
- Plug in the tube of the external suction (7)
- Open the hexagon screw (9) in the stand (10) with the attached key (14) and adjust the working angle
- Close again the hexagon screw (9)

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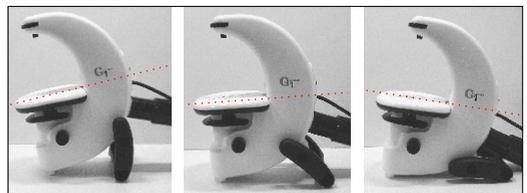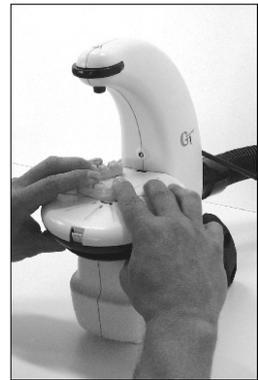
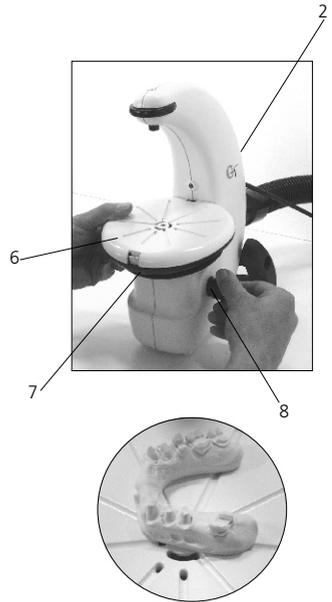
6. Operating the unit

Adjusting the drill depth gauge:

- Press operation plate upwards until the top - the unit must be switched off!
- Adjust the drill depth gauge by using the adjusting screw (8) at the side of the unit
- Release again the operation plate.

Drilling:

- Turn on device at the main switch (2) in the back of the G1-Concept.
- Turn on the suction device
- Place your model on the drilling table. Drilling position concerning the laser point.
- Fix the model with both hands and press the operation plate (7) slowly upwards until the top
- Release again the operation plate.
- By the use of the suction device you get a precise and clean pinhole.

6.1 Security advice

- Never change the tool when the device is switched on!
- For the change of the tool it's essential to use the finger protection!
- Operate G1-Concept never if hand or finger are in the area of the laser line!
Risk of injury!
- Pinhole drilling pursue absolutely with corresponding dust suction!

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7. Change of the drill

- Switch off mains switch
- Press operation plate (7) upwards
- Remove old drill - use finger guard!-
- Insert new drill
- Max. tool length 34.0 mm!
- Release operation plate into zero position
- Adjust the drilling depth gauge (like in chapter 6)
- Switch on the unit
- Unit is now prepared for processing

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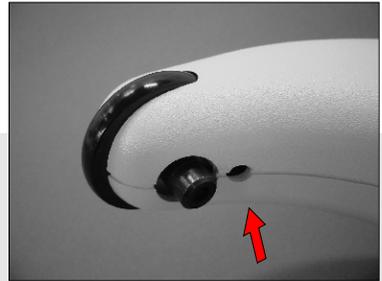
(Abb. / Pic. 1)

8. Maintenance and care

- Clean device occasionally with a soft cloth.
- When using without suction open Cover of the drilling table and vacuum the inside aerea occasionally.
- Adjusting of the laser pointer by opening the hexagon screw with hexagon key SW03. After positioning close screw again (Pic. 2 - 4)
-  Further maintenance is not necessary!
- Never use compressed air or cleaning agent!

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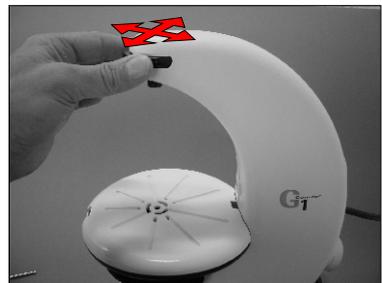
9. Possible faults

Net switch lighting and net LED are not working:

- > net fuse(3,15 A/T 220V No.1396)
(6,3A/T 100V-115V No.3107)
- check and exchange when required!

By all other faults send the device directly to SCHICK-Dental or an authorized dealer

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This unit complies with the current VDE (German association of electrical technicians) regulations concerning safety and suppression.

These operating instructions should be readily accessible and are best kept close to the unit itself.

We would like to take this opportunity to advise you that a proper repair service and suitable qualified personnel are required for such highly developed technical equipment.

SCHICK guarantees to carry out perfect repairs using original spare parts and we are extending another **6 month guarantee** on repairs in our house

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Nous, GEORG SCHICK DENTAL GmbH
Georg Schick Dental GmbH