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**Operating Instructions**  
**Model saw G2 Concept**

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We are pleased that you decided to buy a highly developed piece of equipment from SCHICK and would like to wish you every success when working with your new G2 Concept all purpose model saw.

We wrote these operating instructions to enable you to get accustomed to your new piece of equipment and to provide you with the correct operating and maintenance instructions.

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Index	Page:
1. List of contents - accessories	4
2. Range of applications	5
3. General information Caution !	6
4. Picture G2 Concept - operating elements	7
5. Setting up	8
6. Operating the unit	9
7. Exchanging the saw blade	10
8. Calibrating laser beam	11
9. Maintenance and care	12
10. Possible faults	12
11. Technical data	13
12. Declaration of conformity	14

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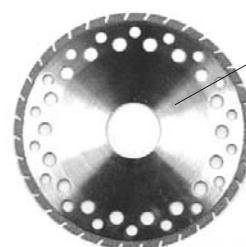
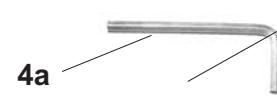
6

5

## 1. List of contents - accessories/spare parts

G2 ConceptArt.No.: 1500

Model saw complete with line marking laser (1)

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Laserklasse: 3A

## **2. Range of Applications**

Until now, many models have necessitated saw cutting by hand, due to the limited space between prepared teeth or irregularities in the alignment of the plaster dies. With the new G2 Concept, these sawing operations can be carried out mechanically with highest precision, because the G2 Concept is operating from below. The G2 Concept provides for the optimum saw cutting of all models without damaging the preparation lines.

Soft starting, instant stopping and two-hand sensor technology are guarantee for your.

Conditions of environment: - interior 10° - 40°C

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WEEE-Reg.-Nr. DE 78620387

### **3. General Information**

- 3.1 Ascertain that your mains supply coincides with the data in the rating plate.
- 3.2 The model saw G2 Concept is not suitable for the following applications:
  - in areas where there is a risk of explosion
  - on patients
  - for working on moist materials
- 3.3 Insure that all regulatory requirements are observed during use
  - always wear protective glasses
  - take care for good illumination
  - use dust suction

### **Caution**

- Use only with original SCHICK saw blade
- Repairs and other technical procedures must only be carried out by suitable qualified personnel, authorized by SCHICK.
- SCHICK do not guarantee the model saw G2 Concept should it not have been used in accordance with the operation instructions.

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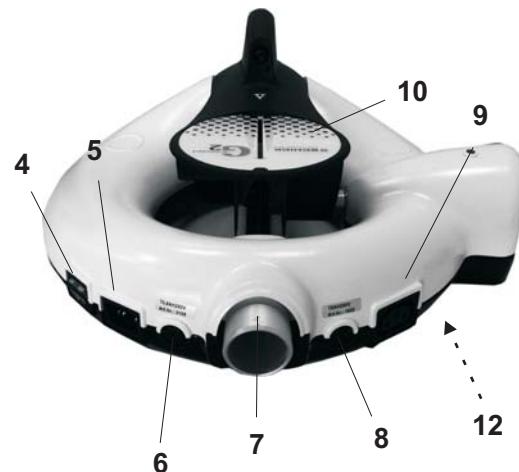
#### 4. Picture G2 Concept

- operating elements


- 1 - hand rest
- 2 - saw table
- 3 - control lamp
- 4 - mains switch
- 5 - socket for mains cable
- 6 - fuse 800 mA T/230V (2A T/100V/115V)
- 7 - extractor socket for external  
dust suction
- 8 - fuse motor 8 A T
- 9 - socket to plug in external  
dust suction
- 10 - line-marking laser
- 11 - saw blade

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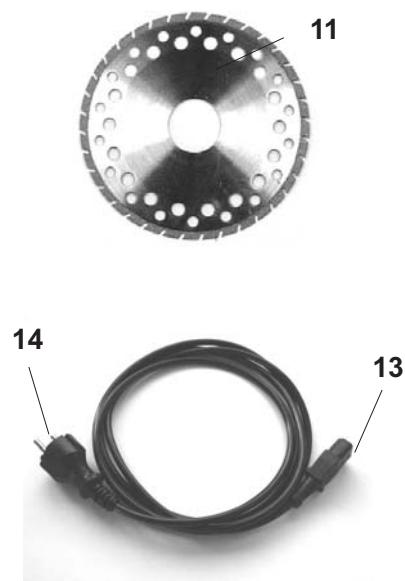
## 5. Setting up

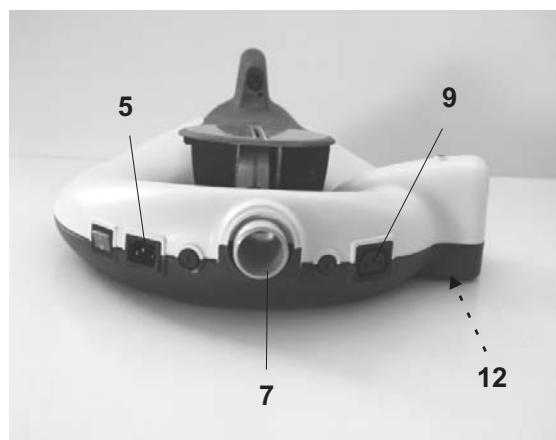Unpack the G2-Concept and check if complete. Take off carefully transportation lock (15) for model table and saw-blade. Check whether power data with the information on the identification plate (12) corresponds. Mains cable (13) in the mains socket (5) and afterwards power plug (14) in mains socket or in the external suction socket. Plug in the tube of the external suction (7). In use of an external suction without controller plug mains cable of the external suction with an adapter in the G2-Concept socket (9).

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## 6. Operating the unit

Turn on device at the main switch in the back of the G2 Concept.

To be sawed press the model with the Pins downwards into the supporting plate (according to the length of your pin system use 1 or 2 supporting plates) Put model with supporting plates on the saw table and position with both hands. Pay attention that both wrists lie on the sensors and take thumb under the saw table and fix the model with forefinger and middle finger on the model table. Align position in the laser line accurately. Press model easily downwards. The saw and the external suction will be turned on automatically in operation by the inserted sensor technology. Press model under light pressure farther downwards. Now the saw blade dives from below by the supporting plate and into the plaster arch and produces a clean saw cut in parallel to the pins. If the optimal cut depth is reached, drive back the saw table slowly again in the starting position. Then the saw switches off itself again automatically. To work nearly dust free the external suction will run a few time longer and stop then automatically.

## 2. Security advice

Operate saw never if hand or finger are in the area of the laser line!

Risk of injury! Saw pursue only with protection eyeglasses! Saw

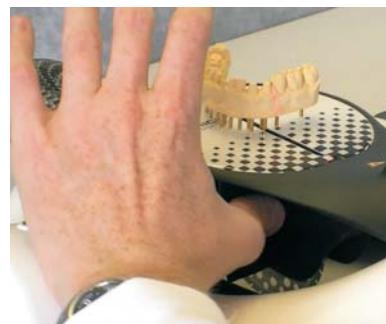
pursue absolutely with corresponding dust suction!

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
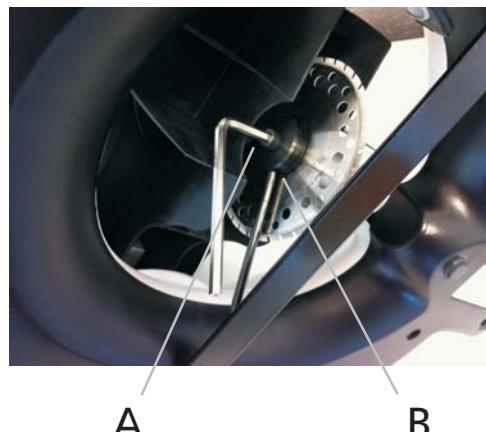
## 7. Change of the saw blade

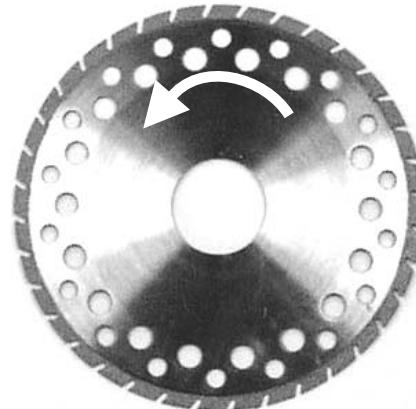Pull net plugs.  
Decrease suction tubes.  
Turn G2 Concept and place preferably on the high thighs.  
Unscrew inside hexagonal screw (A) and remove fixation ring  
(use counter pressure with the provided tool in the intended  
whole (B) on the shaft).  
Exchange saw blade (Use only original SCHICK diamond discs!)  
Insert fixation disc again and screw in the inside hexagonal  
screw under counterholds with the intended tool.

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## 8. Calibrating the laser beam of modelsaw

G2 - Concept (as of series number 150.416)

1. Switch the unit on.

2. Use allen key 3mm (pic.1) horizontally and undo clamp bolt.

ATTENTION: Avoid direct eye contact with the laser beam!

3. Use allen key 2,5mm with an ball-had sloping (pic.2) into levelling screw placed below the clamp bold.
4. When screwing in the levelling screw (pic.2) the laser beam moves to the right, when screwing out the levelling screw the laser beam movesto the left.
5. Adjust laser beam in the desired position and secure clamp bolt (pic.1) by using allen key 3mm.

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Attention  
Do not look directly  
into the laser beam source!

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f.e.: motion of laser  
beam to the right side

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## 11. Technical Data

Dimensions:	Width: 375mm Height: 160mm Depth: 390mm
Weight:	4,5 kg
Power supply:	230 / 115 / 100 V
Nominal frequency:	50/60 Hz
Power input:	130 W
Speed range:	7.500 min-1
Diamond saw blade:	Ø 85 x 20 mm
segmented	
Width of cut:	0,3 mm
depth of cut:	26,5mm
Laser class:	3A
Safety class:	2

This unit complies with the current VDE (German association of electrical technicians) regulations concerning safety and suppression.

These operating instructions should be readily accessible and are best kept close to the unit itself.

We would like to take this opportunity to advise you that a proper repair service and suitable qualified personnel are required for such highly developed technical equipment.

SCHICK guarantees to carry out perfect repairs using original spare parts and we are extending another 6 month guarantee on repairs in our house.

Subject to technical modification without prior notice

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declare herewith, that the product

Model sawing machine G2 Concept

is in conformity with the following provisions of Directive:

- 2001/95/EG (general product safety)
- 2006/42/EG (machinery directive)
- 2006/95/EG (low voltage directive)
- 2004/108/EG (EMV directive)

Name and address of Wolfgang Schick  
person in charge: Lehenkreuzweg 12  
88433 Schemmerhofen

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
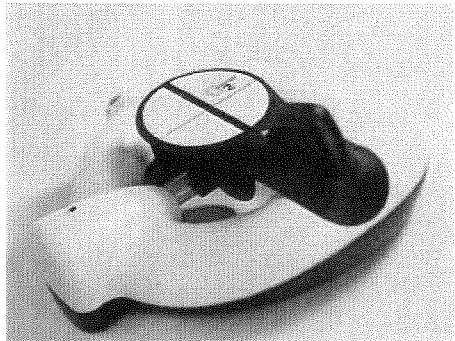
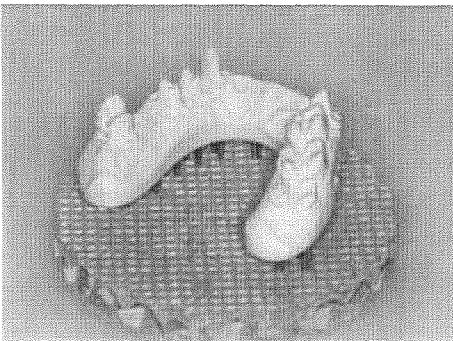
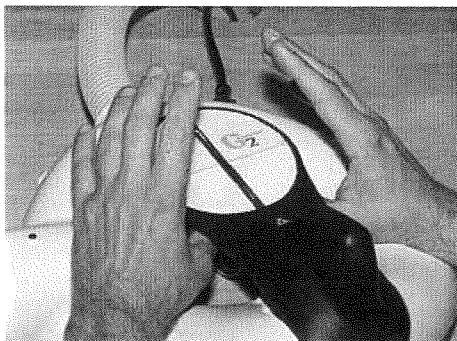
The thumbs are placed on both sides under the  
saw table...

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
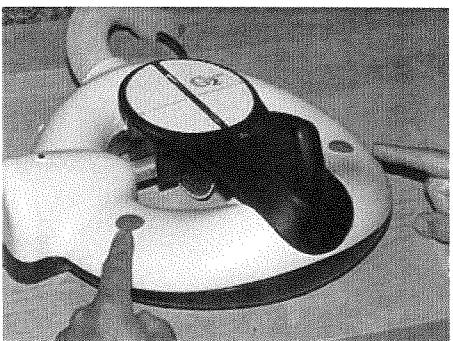
... the forearms must be placed on the two  
attempt sensors on both sides.

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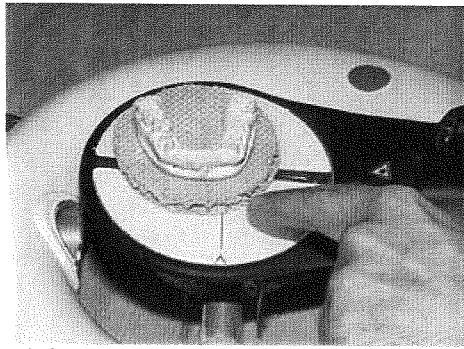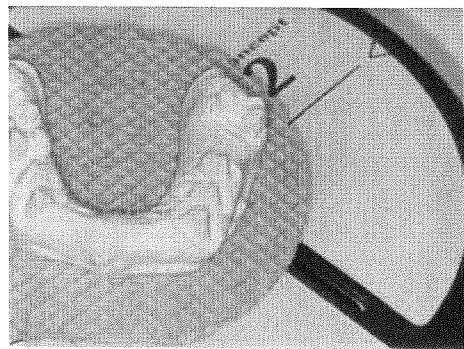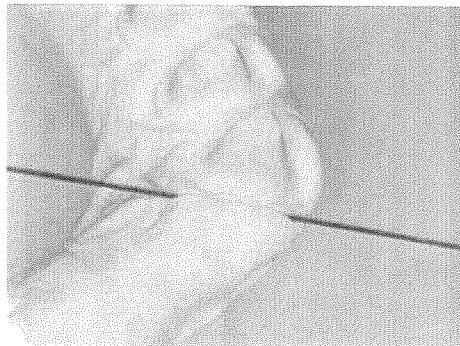
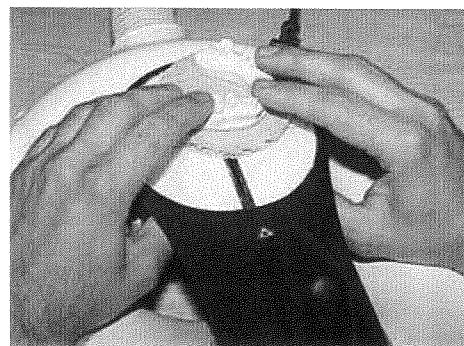
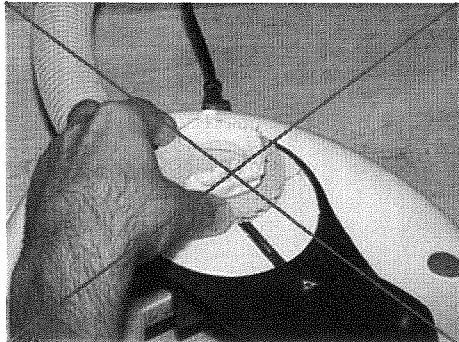
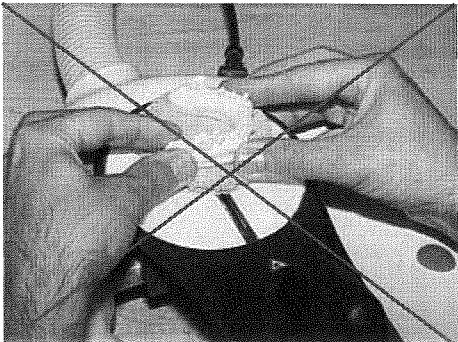
"Negative example"

plaster arche only with one hand fixated -- one hand  
operation is not possible for safety reasons!

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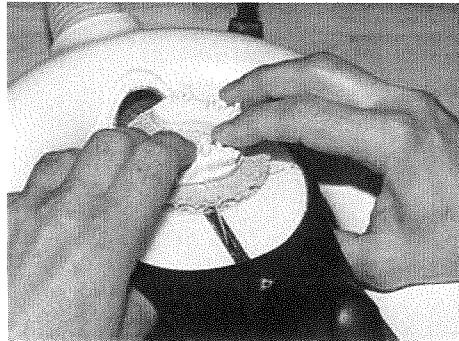
"Negative example"

the thumbs are not in the protected area under the saw  
table, wrists are not placed on the sensors!

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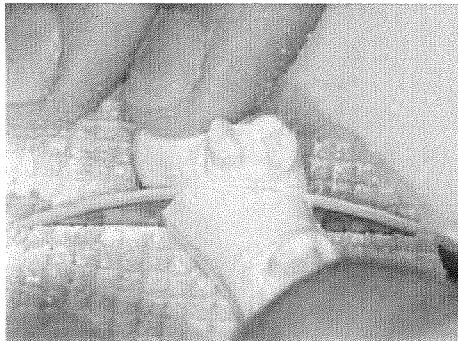
"right application"

align plaster arche with the laser light and press down the  
saw table slowly with a low pressure

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
The cut can be completely carried out or execute till approx.  
1-2 mm below the plaster limit.

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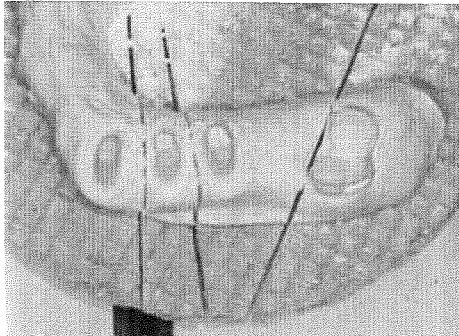**After sawing the complete model the individual segments can be got out easily. The break will always be carried out in the thinnest point.**

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Georg Schick Dental GmbH