

# VACALON



## Pressable Ceramic Disposable Plungers



### **MARKER ON TOP!**

 **NO PRE-HEATING OF PLUNGER!**

Suitable for all pressable ceramic systems.



### **MARKIERUNGSPUNKT OBEN!**

 **NICHT VORWÄRMEN!**

Passend für alle Presskeramiksysteme.



### **RÉFÉRENCE AU-DESSUS!**

 **NE PAS PRÉCHAUFFER!**

Convient pour tous systèmes de céramique pressée.



### **EL PUNTO TIENE QUE ESTAR ARRIBA!**

 **NO PRECALIENTAR EL PISTÓN!**

Indicador por todas las cerámicas de pressofusión.



### **LASCIARE IL PUNTO OIL ALTO!**

 **NON PRERISCALDARE LO STANTUFFO!**

Indicato per tutte le ceramiche pressofuse.

## GENERAL INFO

- **DO NOT PREHEAT PLUNGER.** Insert plunger, with the dot on the outside, into ring on top of ingot before placing in the pressing furnace.
- The sign of slight cracking at the contact area of the plunger and ingot is normal due to differences in expansion. These slight cracks are not transferred to the pressed restoration due to the efficient heating and cooling of the plunger.
- Avoid inhaling dust. Use an approved dust mask. Overexposure may cause delayed lung injury.
- Avoid contact with eyes. Wear safety glasses for protection during cutting or polishing. In case of contact with eyes, flush eyes with water and seek medical attention.

## AVAILABLE SIZES

Item: 40001 12 mm dia. for 2 gram ingots

Item: 40002 13 mm dia. for E-Max style ingots

Item: 40003 16 mm dia. for 5 gram ingots

## HOW TO REORDER

**800.729.8192** us **614.577.1945** international 614.759.4999 fax

sales@vacalon.com e-mail www.vacalon.com

Vacalon, 12960 Stonecreek Dr, Ste C, Pickerington, OH 43147

Made in the USA